



ROTHBAND

RADIATION PROTECTION

INSTRUCTIONS FOR USE

MULTI USE TABLE SCREEN

VERSION 2.0

**UK
CA CE**

Table of Contents

INTRODUCTION	3
INTENDED USE	3
REGULATORY COMPLIANCE	3
CAUTIONS AND WARNINGS	4
COMPONENTS	5
CLEANING INSTRUCTIONS	5
Lead Curtain	5
Metal Framework	5
SPECIFICATIONS	5
WARRANTY	6
CONTACT DETAILS	6

INTRODUCTION

The Multi use table Screen is made from flexible lead vinyl sheet with two handles which are design to fit a range of DIN rail sizes. The handles can be positioned in a number of different places along the screen to allow them to be fully configurable to a range of surgical tables. It is available in 0.5mm Pb or 1mm Pb, depending on requirements.

PRODUCT CODES

TS03

INTENDED USE

For use as protection against secondary radiation. Only to be used after guidance from a qualified RPA and in compliance with relevant local rules.

REGULATORY COMPLIANCE

The Wipeable foam positioning pad range complies with MDR (EU) 2017/745 (EU) and MDD 93/42/EEC (UKCA) concerning medical devices (class I) and has been designed and manufactured to comply with the following standards:

Standard	Description
EN ISO 61331-1 2014 BBG*	Protective devices against diagnostic medical X-radiation. Determination of attenuation properties of materials.
EN ISO 14971:2012	Medical devices - Application of risk management to medical devices (ISO 14971:2007, Corrected version 2007-10-01)
EN ISO 15223-1:2016	Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements.

CAUTIONS AND WARNINGS



- Only to be used after guidance from a qualified RPA and in compliance with relevant local rules.
- Should be part of your regular testing programs as per lead aprons.
- Do not use if any obvious tears or cracks are apparent.
- Employ caution when moving over uneven floor surfaces.
- Lower the top screen to its lowest point before transport.
- Do not allow sharp objects to penetrate the lead curtain.
- The above warnings & cautions and any safety considerations should be observed on a routine and regular basis, not only upon installation.


COMPONENTS



LEAD PANEL



ADJUSTABLE HANDLE

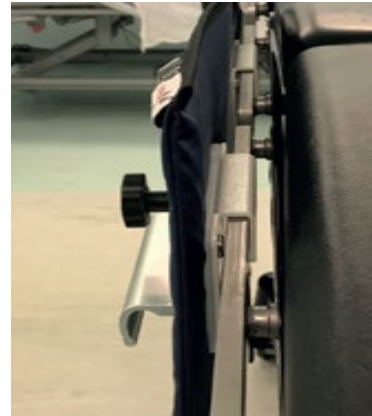

DIN ATTACHMENTS

CLEANING INSTRUCTIONS


Lead Curtain

The outer covers may be cleaned using mild disinfectant or detergent. Strongly acidic, alkaline, alcohol/phenol based, or abrasive products should not be used. DO NOT LAUNDER. The product cannot be sterilised.

Metal Framework

Units can be cleaned with water and normal cleaning solutions. After cleaning, ensure the unit is dry before use.

SPECIFICATIONS

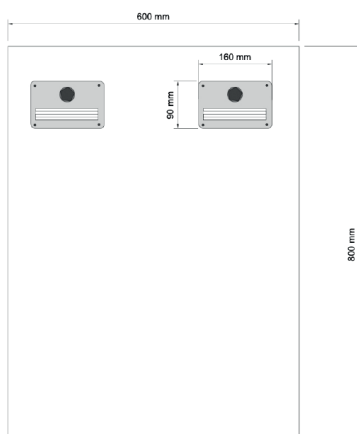

Lead Equivalence mm Pb	0.5, 1.0
Options	Single Panel / 200mm Slats
Dimensions	850mm x 600mm (Other options available)

WARRANTY

This product comes with a 12 month warranty

CONTACT DETAILS



WSR Medical Solutions
(Trading as Rothband)
Unit 2 Hargreaves Mill
Hargreaves Street
Haslingden
Lancashire
BB4 5RQ
UK

Tel: 01706 830086

e-mail: sales@rothband.com